

# TCS<sup>®</sup> All Purpose Hydrocolloid

**High quality duplicating material**  
for accurate, consistent results

## 2.7 Gallon

- High tear strength
- Ideal for stone, investment and pour acrylics
- High precision to capture fine details
- Long-lasting
- Quick melt down time



**Save time and money by using TCS<sup>®</sup> Hydrocolloid in the TCS<sup>®</sup> Dispenser.**

**Built Tough!**

**Machines built in 1966 still in use!**

- Available in 2, 3 and 10 gallons
- Double boil out tank
- Automatic melting and cooling
- No water connections required
- Double the life of hydrocolloids
- Special dispensing valve eliminates clogging



Made in the U.S.A.  
5 year warranty

**Distributed by:**

**Imperial Trading  
72-59 Kessina Blvd.  
2nd Floor  
Flushing, NY 11367  
718-793-8846**



**IMPERIAL TRADING LTD.**

72-59 KISSENA BLVD , FLUSHING , N.Y. 11367

PHONE (718) 793-7490 (800) 733-7490 FAX (718) 793-9831

**Visit Us At:**<http://www.imperialdental.net> **Email:**[imperialdental@hotmail.com](mailto:imperialdental@hotmail.com)

---

## Instructions for TCS® Hydrocolloid

- Crush or cut in small fragments.
- Melt using indirect flame, avoid exceeding 195°F.
- If you use a procelainized container (pewter) pour it into the jar at 130°F.
- Let the plaster model stand in water for 20 minutes at 105°F
- Fill the flask and let it cool at room temperature for 20 minutes.
- Once cooled place in water (40°F) for 20 minutes.
- Remove from the model, observe the details and remove the pernicious fragments.
- Fill with the compound over a vibrator.